Alpha α 101

Alpha α 101

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*For an efficient breathing and inhalation
therapy in hospital*

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CE 0123

Areas of application

Combinations of suitable methods of physiotherapy and apparative medicine are to be chosen for the treatment of intensive-care patients suffering from imminent or manifest respiratory insufficiency. Today, assisted respiration is the standard praxis for the supply of patients before and after operations. Another important field is the prevention of respiratory failures in the context of modern intensive therapy as, for example, during severe general diseases or regularly after larger surgical thorax operations. Assisted overpressure respiration in combination with simultaneous aerosol administration is a possible treatment of cooperative, responsive patients suffering from respiratory depression. Assisted respiration may also be a supporting means in cases of manifest atelectasis. The application of intermittent overpressure respiration is made even more efficient by the possibility of adjusting oxygen concentrations to the patient's needs.

Function

Air pressure and oxygen are supplied via an external source. The machine functions are controlled by a microprocessor. The following data is displayed: set values, airway pressure, breathing frequency, and duration of treatment. The device has an automatic break function. By his or her inspiratory efforts, the patient generates a negative pressure that starts the inspiration phase of the device depending on the trigger point set. When the preset airway pressure is reached, the device switches over to expiratory mode. The expiration phase can be delayed by an adjustable expiration resistance. During the inspiration phase, a jetstream nebulizer is supplied with compressed air allowing for an intermittent aerosol therapy. The gas flow during inspiration can be adjusted to individual patients. Oxygen concentrations can be set to and kept constant at 30, 40, and 50% within very small margins and independent of the inspiratory flow.

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